

## SWASH<sup>®</sup> GOLD WIPES – FRAGRANCED

### GENERAL INFORMATION

Article number	Bo4028-8
Basic UDI-DI	PP10417BODY.WASH95
UDI-DI	111666460136
Intended use	Intended for a complete body wash of incontinent and/or bedridden patients as an infection prevention measure.
Content of package	8 wipes
Content of box	44 packages
Manufacturer	Arion Manufacturing B.V. – The Netherlands Rutherford 11, 6422 RE Heerlen
Issue date	07/2020
Revision date	N.A.


### PRODUCT INFORMATION

Type of product	Medical device class 1 (2017/745/EC)
Composition cloth	Polyester & Lyocell
Production method cloth	Needle punch
Thickness cloth	ca. 110 GSM
Box weight	ca. 4775 gram
Total net waste weight cloths	ca. 35.2 gram
Total net waste weight foil	ca. 5 gram
pH-value	5 – 6
Dimensions cloth	20 cm x 20 cm
Dimensions package	10 cm x 26 cm

### INGREDIENTS LOTION

	PURPOSE	%
Aqua	Solvent in which the other ingredients are dissolved.	75-100
Propylene Glycol	Water-attracting ingredient that reduces dehydration of the skin. Next to that, it is a solvent and it determines the viscosity of the lotion.	1-5
Glycerin	Water-attracting ingredient that reduces dehydration of the skin. Also used as a solvent in many cosmetic products like shampoos.	1-5
Caprylyl Glycol	Skin conditioner. Also has preservative properties.	0.1-1
Polysorbate 20	Emulsifier. Binds the lotion and stabilizes the emulsion.	0.1-1
Disodium Cocoamphodiacetate	Mild cleaning ingredient with skin treating properties.	0.1-1
Tocopherol (Vitamin E)	Preservative and moisturizer to prevent dehydration of the skin.	0.1-1
Glyceryl Caprylate	Humectant. Also has preservative properties.	0.1-1
Potassium Sorbate	Natural preservative.	0.1-1
Dipropylene Glycol	Solvent for the preservative system.	0.1-1
Parfum	Mixture of fragrance ingredients	0.1-1
Sodium Chloride	Additive.	<0.1

## INSTRUCTION FOR USE

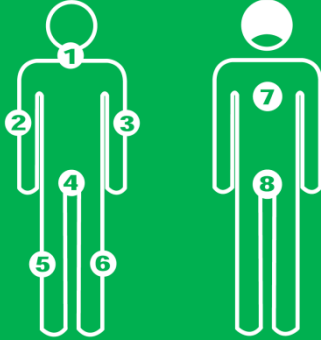

Use 1 wipes for each part of the body, as indicated below:

1. Face, neck and chest
2. Right arm and armpit
3. Left arm and armpit
4. Lower body front
5. Right leg and foot
6. Left leg and foot
7. Back
8. Lower body back

If necessary, heat the Swash<sup>®</sup> package before use.

Check the temperature of the wipes of a heated package before use.

The wipes are disposable and each package should be used for one person only.

Skin creases must be patted dry using a towel to prevent moisture-related skin lesions.

Only suitable for external use.

Nursing staff is advised to use up a packet of Swash within 48 hours after opening.

Do not flush down the toilet.

## TEST RESULTS

Eye irritation test

Results show that the lotion does not cause any irritation of the cornea and the iris. In addition, the conjunctiva do not swell or turn red.

Skin irritation test

Results show that the Swash<sup>®</sup> Gold Wipes lotion does not cause any irritation of the skin. There was no sign of edema or red skin during or after the test.