

## SWASH<sup>®</sup> PERINEUM<sup>+</sup> WIPES – FRAGRANCE FREE

### GENERAL INFORMATION

Article number	K04540-8
Basic UDI-DI	PP10417INCONTINENCE72
UDI-DI	111666473633
Intended use	Intended for perineal care of incontinent patients as a measure to prevent skin damage associated with exposure to urine or stool (IAD) or as a measure to manage such skin damage (IAD) in an early stage.
Content of package	8 wipes
Content of box	40 packages
Manufacturer	Arion Manufacturing B.V. – The Netherlands Rutherford 11, 6422 RE Heerlen
Issue date	07/2020
Revision date	N.A.

### PRODUCT INFORMATION

Type of product	Medical device class 1 (2017/745/EC)
Composition cloth	Polyester & Lyocell
Production method cloth	Spunlace
Thickness cloth	ca. 82 GSM
Box weight	ca. 10000 gram
Total net waste weight cloths	ca. 35 gram
Total net waste weight foil	ca. 5 gram
pH-value	5 – 6
Dimensions wipe	20 cm x 25 cm
Dimensions package	10.5 cm x 25.5 cm


### INGREDIENTS LOTION

### PURPOSE

		%
Aqua	Solvent in which the other ingredients are dissolved	75-100
Dimethicone	Water-attracting, film-forming ingredient that reduces dehydration of the skin by preventing evaporation of water and works as a skin protectant. Also used as an anti-foaming ingredient.	3
Propylene Glycol	Solvent that also works as a water-attracting ingredient, thus reducing dehydration of the skin.	1-5
Magnesium Aluminum Silicate	Natural smectite clay that helps to stabilize the oil-in water emulsion.	1-5
Glycerine	Natural moisturizer that makes water available to the skin cells for better hydration of the skin.	1-5
Caprylic / Capric Triglyceride	Naturally occurring fatty acids that function as emollients (ingredients that soften your skin).	0.1-1
Lauryl Glucoside	Plant-derived, very mild skin cleansing ingredient	0.1-1
Sodium Lauroyl Lactylate	Natural mild emulsifier that helps to create the oil-in-water emulsion.	0.1-1

Polyacrylate Crosspolymer-11	Hydrogel-forming thickening agent for oil-in-water emulsions.	0.1-1
Glyceryl Stearate	Emulsifying wax that acts as a skin lubricant.	0.1-1
Caprylyl Glycol	Skin conditioner that also has preservative properties.	0.1-1
Cetearyl Alcohol	Plant-based fatty alcohol that helps to create the oil-in-water emulsion and improves the spreadability of the lotion.	0.1-1
Glyceryl Caprylate	Humectant. Also has preservative properties.	0.1-1
Sodium Stearoyl Lactylate	Natural mild emulsifier that helps to create the oil-in-water emulsion.	0.1-1
Sodium Hyaluronate	Polysaccharide that is part of the intracellular matrix of the skin. Supports skin renewal in irritated skin areas.	0.1-1
Tocopherol (Vitamin E)	Anti-oxidant and moisturizer	0.1-1
Potassium Sorbate	Natural preservative. Also functions as pH-stabilizer.	0.1-1
Dipropylene Glycol	Solvent for the preservative system	<0.1

## INSTRUCTION FOR USE

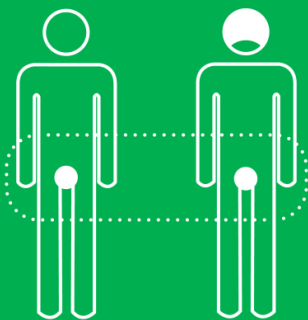

Clean the soiled perineum area from front to back using a sufficient number of wipes.

Swash® Perineum+ Wipes are intended to clean, care for, protect and to help to restore the skin.

Ensure that the skin is dry before replacing the incontinence material and/or underwear.

If preferred, the pack of wipes can be preheated in a microwave oven.

Always check the temperature before use.

Each pack should only be used on a single care recipient.

Avoid contact with the eyes.

Where the care recipient suffers from persistent redness and severe wounds (no irritation, but badly damaged skin) please consult with a wound consultant, dermatologist, dermatology nurse, GP or a geriatric specialist.

The wipes are disposable and each package should be used for one person only.

Only suitable for external use.

Nursing staff is advised to use up a packet of Swash® within 48 hours after opening.

Do not flush down the toilet.

## TEST RESULTS

### Skin irritation test

Results show that the Swash® Perineum+ Wipes lotion does not cause any irritation of the skin. There was no sign of edema or red skin during or after the test. Nor is there any indication it causes allergic reactions.