

INSTRUCTIONS FOR USE

SWASH[®] CLEANSING WIPES 48 & 24-pack

Product code: L05012-48 (UDI-DI: 08717127938216) / M05012-48 (UDI-DI: 08717127938223)
/ M05012-24 (UDI-DI: 08720663610485)


User: Professional or health care provider

Intended purpose:

Swash[®] Cleansing Wipes are intended for incontinence cleanup of incontinent and/or bedridden patients as an infection prevention measure.

Supporting materials:

- Disposable gloves
- Rubbish bin
- Incontinence material (if necessary)
- Clean clothing and / or linens (if necessary)

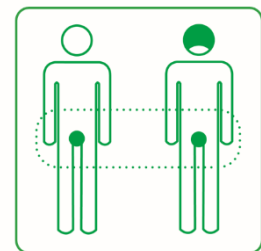

Instructions for use:

Clean the soiled area with a sufficient number of wet wipes. Use the re-sealable label to close the packaging after each use. This ensures that the wipes do not dry out. Use each package for one person only. See below for detailed procedure.

Detailed procedure:

The Swash[®] Cleansing Wipes can be stored on the ward in a central place or next to the bed of the care recipient.

- 1 Before starting, wash your hands and put on a pair of disposable gloves.
- 2 Tell the care recipient what you are going to do and keep communicating throughout the washing procedure.
- 3 Have the care recipient lie in a supine position with a pillow under his / her head. Have the care recipient remove his/her outerwear; if necessary, help the care recipient with this.
- 4 Remove wet and dirty incontinence materials.
- 5 Open the package of Swash[®] Cleansing Wipes and take out a wipe. Wash the genitals with a Swash[®] Cleansing Wipe. Use as many wipes as necessary. The skin dries by itself through the evaporation of the lotion. If necessary, dry the skin folds by blotting with a tissue paper.
- 6 Ask the care recipient to turn onto his / her side and offer support if necessary. Clean the soiled perineum region (the buttocks and coccyx) of the care recipient with a Swash[®] Cleansing Wipe. Use as many wipes as necessary for the removal of all the faeces and / or urine from the skin. Preferably, place the used Swash[®] Cleansing Wipes within the incontinence material.
- 7 Inspect the perineum region for Incontinence Associated Dermatitis and other skin condition variables.
- 8 If the care recipient is at risk of getting Incontinence Associated Dermatitis (IAD) or in case the IAD needs to be treated, continue with the protocol of *Swash[®] Perineum Gloves/ Swash[®] Perineum Wipes* or *Swash[®] Perineum+ Wipes*.
- 9 Give the care recipient new, clean and dry incontinence material. Change the bedding if wet or soiled and dress the care recipients again and put him / her in the desired position.
- 10 Throw the empty package of Swash[®] and the wipes in the bin. DO NOT FLUSH Swash[®] Cleansing Wipes down the toilet!
- 11 Report any changes in the condition of the skin after the perineum washing.
- 12 Repeat this procedure after every urinary or faecal incontinence episode.

ATTENTION:


- At all times, make sure to take the feelings, comfort and privacy of the patient into consideration.
- Each package is to be used for only **one** person. The remaining washcloths may be used for the same care recipient.
- Only suitable for external use.
- **Do not flush the washcloths down the toilet. Throw the empty package of Swash® and the wipes in the bin.**

CONTRA-INDICATIONS:

- If the care recipient refuses the use thereof.
- If it is known that the care recipient is sensitive or allergic to one or more of the ingredients.
- On non-intact skin & open wounds.



PICTOGRAMS IN PACKAGE:

	Temperature limit		Use within 1 month of opening
	Keep away from sunlight		Throw to the garbage bin
	Keep dry		Do not macerate
	Do not use if package is damaged and consult instructions for use		Product free of latex
	Do not re-use		Produced in a clean room
	Non-sterile		Product does not contain fragrance
	Medical Device		Product contains fragrance
	CE-marking		Do not flush
	Batch code		Consult instructions for use or consult electronic instructions for use
	Manufacturer		
	Use by date		
	Unique device identifier		

The information in this protocol specifically relates to the use of Swash® Cleansing Wipes for cleaning the perineum region of incontinent care recipients. Additional interventions and steps may be added to this protocol according to other protocols and guidelines in your healthcare facility.

Arion Manufacturing B.V. • Rutherford 11 • 6422 RE Heerlen, the Netherlands
+31 (0)45 209 4000 • manufacturing@arion-group.com • www.arion-group.com