

INSTRUCTIONS FOR USE

SWASH[®] REFRESHING WIPES 32-pack

Product code: A04010-32 (UDI-DI: 08717127937110) / B04010-32 (UDI-DI: 08717127937127)

User: Professional or health care provider

Intended purpose:

Swash[®] Refreshing wipes are intended for a hand/face wash of incontinent and/or bedridden patients as an infection prevention measure.

Supporting materials:

- Facilities for hand hygiene of care provider
- Disposable bag if required
- Rubbish bin

Instructions for use:

Use sufficient wipes to clean the soiled area. Use the resealable label to close pack after each use. This ensures that the wipes do not dry out. See below for detailed procedure.

Detailed procedure:

Place the Swash[®] Refreshing Wipes centrally on the ward. Do not store the package in warm places.

- 1 Before handing out the wipes, it is important that the caregiver realizes proper hand hygiene. This prevents that the package will be contaminated.
- 2 Inform the care recipient what the cloth is intended for.
- 3 The caregiver distributes the wipes on the ward, one wipe per care recipient. Make sure that only the caregiver holds the package in his hands and that the person who removes the wipes from the package has applied a proper hand hygiene.
- 4 If possible, the following method of hand hygiene shall be used. This ensures an optimal cleaning of the hands.



- 5 After a few minutes the wipes are collected again (possibly with a disposable bag) by the caregiver and then discarded in the bin.
- 6 If the caregiver has been in contact with the dirty wipes when collecting them, it is recommended that hand hygiene is again realized.

ATTENTION:


- The wipes are for single use only.
- It is advisable to use up an opened package within 48 hours.
- Only suitable for external use.
- Be careful when washing the eyelids.
- If skin irritations appear when using Swash® Refreshing Wipes, discontinue use and consult a physician if necessary.
- **Do not flush the wipes down the toilet. Throw the empty package of Swash® and the wipes in the bin.**

CONTRA-INDICATIONS:

- If the care recipient refuses the use thereof.
- If it is known that the care recipient is sensitive or allergic to one or more of the ingredients.
- On non-intact skin & open wounds.



PICTOGRAMS IN PACKAGE:

	Temperature limit		Use within 48 hours of opening
	Keep away from sunlight		Throw to the garbage bin
	Keep dry		Do not macerate
	Do not use if package is damaged and consult instructions for use		Heating advice
	Do not re-use		Product free of latex
	Non-sterile		Produced in a clean room
	Medical Device		Product does not contain fragrance
	CE-marking		Product contains fragrance
	Batch code		Do not flush
	Manufacturer		Consult instructions for use or consult electronic instructions for use
	Use by date		
	Unique device identifier		

The information in this protocol relates specifically to the use of Swash® Refreshing Wipes for cleaning and caring for users' mouth and hands. Additional interventions and steps may be added to the protocol in accordance with other protocols and guidelines in your healthcare facility.

Arion Manufacturing B.V. • Rutherford 11 • 6422 RE Heerlen, the Netherlands
+31 (0)45 209 4000 • manufacturing@arion-group.com • www.arion-group.com